

## QUICK GUIDE - HOW TO MATCH A DEX TO THE HEARING AID

This quick guide describes how to match a DEX to the hearing aid. The matching procedure can be found in Finalise under DEX matching.

1. The hearing aid has to be connected to Compass <sup>1</sup>.
2. The DEX matching menu is available in the Finalise section.
3. Click on “Start matching” in Compass



4. Hold the DEX 5 cm from the hearing aid and press the arrow up and arrow down button at the same time.
5. When matching is done, Compass will show “Matching complete”.
6. Click on the “OK” button to save the matching.



<sup>1</sup> Before matching an RC-DEX, a battery must first be inserted in the RC-DEX